

ConMed Hyfrecator 2000


Power Cord inlet instructions:



Congratulations on the purchase of your ConMed Hyfrecator 2000. Before you use your Hyfrecator for the first time, you will need to secure your power cord correctly for safe operation.


Your Hyfrecator has a black clamp at the power cord inlet socket. The clamp has three screws attached – pictured below.

Dimple

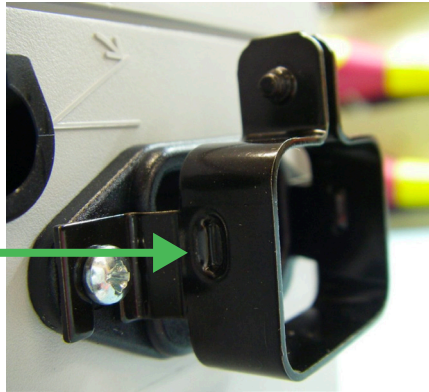

Clamp: The clamp has two dimples inside it on both sides. The purpose of the dimples is to squeeze on to the shoulders of the power cord and secure it within the socket (after the power cord is inserted fully into the socket and seated correctly).

Shoulder

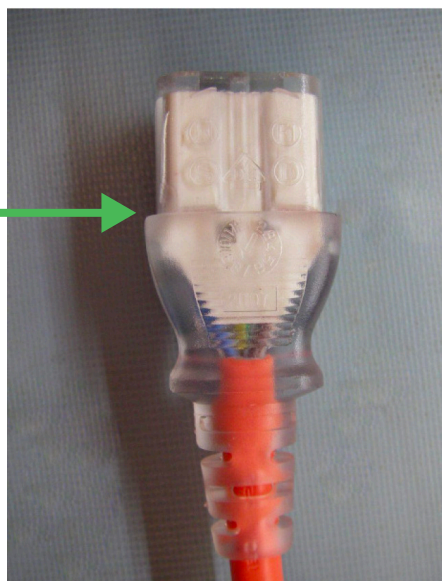

Procedure to secure power cord correctly:

1. Loosen the top screw (black screw) all the way.
2. Loosen both the other screws (important), just a little, about one turn is ok (otherwise the bracket doesn't open wide enough to allow the shoulder to pass through).
3. Insert the mains cable connector in until the shoulder is visible inside the bracket, it should be well seated within the socket.
4. Turn the Hyfrecator 'ON' to verify power.
5. Tighten the screws. Do the side ones first and then tighten the black screw to secure the clamp on the power cord connector.

All Done! ☑

Below is a photo of a fully seated connector.

