



WOUND CARE  
BECAUSE YOU CARE



MOISTURE WICKING FABRIC Ag



Authorised distributor for ActivHeal®  
in New Zealand

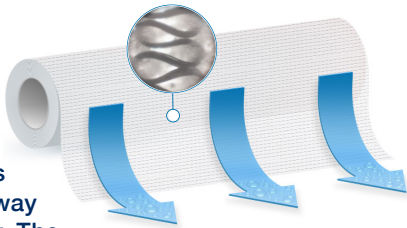
## FEATURES




Fabric acts as a skin protectant // Acts as a lubricant // Reduces friction // Provides a protective environment // Can remain in situ for up to 5 days // Moisture transportation wicks moisture away from skin //

# MOISTURE WICKING FABRIC WITH ANTIMICROBIAL SILVER (Ag) SOFT AND CONFORMABLE FABRIC DESIGNED TO HELP MANAGE SKIN FOLDS AND SKIN-ON-SKIN CONTACT.

## ActivHeal® Moisture Wicking Fabric with antimicrobial Silver (Ag)

is a skin protectant which wicks and moves moisture away from skin to keep it dry. The fabric assists with the reduction of skin on skin friction which provides a protective environment for the skin. The antimicrobial silver reduces colonisation of bacteria and yeasts such as Staphylococcus epidermis, Pseudomonas aeruginosa and Candida albicans within the fabric.



-  **MANAGES MOISTURE**
-  **REDUCES FRICTION**
-  **REDUCES ODOUR**



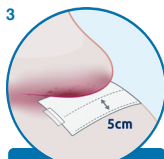


## PERFORMANCE

Kills 99% of bacteria, yeasts and mould within 24 hours (1) Kills 99% of bacteria, yeasts and mould within 24 hours<sup>1</sup>



-  Staphylococcus aureus
-  Staphylococcus epidermis
-  T. mentagrophyte
-  Pseudomonas aeruginosa
-  Candida Albicans

## APPLICATION GUIDE

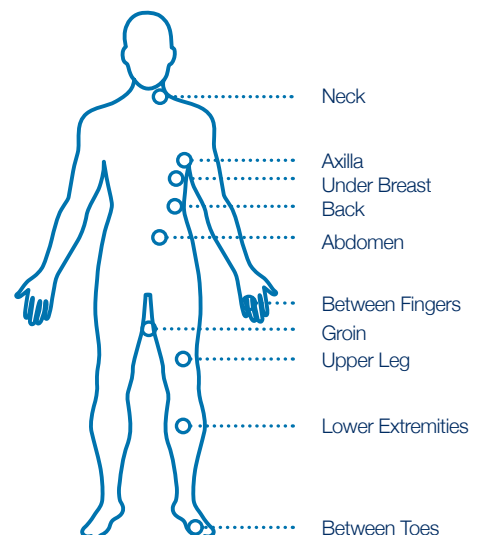
-  **1** Wash and pat dry
-  **2** Cut to required size
-  **3** Lay a single layer of fabric in the skin fold
-  **4** Leave 5cm exposed. Tape if required
-  **5** Separate Skin fold to remove

## SIZES AND CODES

SIZE (CM)	SIZE (INCHES)	QTY PER CARTON	QTY PER CASE	PRODUCT CODE
10x36	23x91	10	6	9024967
10x144 (Roll)	25x366 (Roll)	1	10	9024981

## INDICATIONS

The fabric helps with the management of skin on skin friction, the management of skin folds and other skin on skin contact areas. The soft fabric conforms to anatomical contours and can remain in situ for up to 5 days.



## Advanced Medical Solutions Ltd

Premier Park, 33 Road One, Winsford Ind. Estate,  
Winsford, Cheshire CW7 3RT  
T 08444 125 755 E [activheal@admedsol.com](mailto:activheal@admedsol.com)  
W [www.activheal.com](http://www.activheal.com)



## Capes Medical Supplies

122 Newton Street, Mount Maunganui 3116,  
New Zealand  
T 0800 18 19 19 E [support@capemedical.co.nz](mailto:support@capemedical.co.nz)  
W [capemedical.co.nz](http://capemedical.co.nz)