

# Debrisoft® Pad and Lolly Application



## Step 1

Open the Debrisoft single-use sterile pack.

## Step 2

Moisten the soft, fleecy part of the product with tap water or saline solution, as per protocol.  
Recommended amount:  
Debrisoft pad 10x10cm: 20-40ml  
Debrisoft pad 13x20cm: 90-110ml  
Debrisoft Lolly: 5-15ml  
Always refer to local guidelines. Do not wring or cut.

## Step 3

Gently with controlled pressure, use the soft, fleecy part of the product in a circular motion or long sweeping strokes across the wound bed then outer peri-wound area & surrounding skin.

Use special application techniques shown by trainer.

Use a new Debrisoft for each separate wound/area of skin.

## Step 4

Dispose of the used product in normal clinical waste.

Refer to the IFU for full instructions and precautions.

## Ordering Information

### Debrisoft® Pad

sterile, individually-sealed

Size	REF	Shipping Units (per box/case)
10 cm x 10 cm	34322	5/50
13 cm x 20 cm	136323	5/50

### Debrisoft® Lolly

sterile, individually-sealed

REF	Shipping Units (per box/case)
33224	5/50