

COMFORTABLE **EASY** TO APPLY AND REMOVE

WHAT IS IT?

ADAPTIC® DIGIT Non-Adhering Digit Dressing is made of a unique, tacky, silicone-coated fabric covered with a tubular bandage. It is specifically designed for dressing fingers and toes.



COMFORTABLE

ADAPTIC® DIGIT does not adhere to the wound. It is conformable and non-bulky, therefore allowing maximum digit flexibility.



EASY TO APPLY AND REMOVE

ADAPTIC® DIGIT is quick and simple to apply; no applicator or scissors are required. It removes cleanly from the wound, minimising trauma¹.



ADAPTIC® DIGIT NON-ADHERING DIGIT DRESSING




USER GUIDE

BEFORE APPLICATION

Refer to the instructions for use. Select a size of ADAPTIC® DIGIT according to the size of the finger or toe. Prepare the wound according to appropriate wound management protocol. Ensure skin surrounding the wound is dry. Remove the ADAPTIC® DIGIT dressing from the box.

Dressing preparation

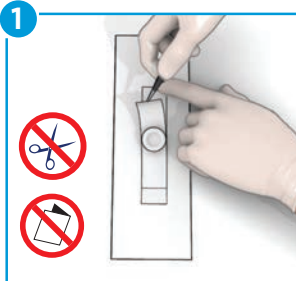


1. Using aseptic technique, peel apart the two sides of the packaging.
2. Remove **ADAPTIC® DIGIT** from shiny backing strip.

Dressing application



1. Place primary silicone non-adhering layer over digit tip and gently smooth over any broken skin.
2. Roll down lower ring of tube bandage over primary strip (and secondary dressing if required).
3. Follow this with the top ring to form a double layered tube for protection.

Dressing change and removal

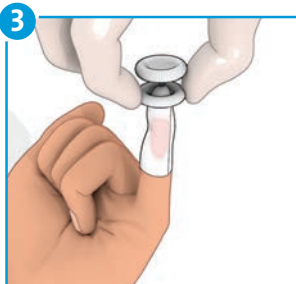

Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound. **ADAPTIC® DIGIT** may be left in place for several days.

1. Gently roll up the upper ring of the tubular bandage and repeat with the other ring.
2. Gently remove the non-adhering layer from the wound bed and discard dressing.

Also available for toes



WHEN TO USE

ADAPTIC® DIGIT dressing is indicated to manage a range of mild to severe digit injuries, such as:

- Lacerations and abrasions
- Traumatic injuries
- Nail extractions
- Suture lines

ADAPTIC® DIGIT can be used both as a primary and secondary dressing².

Good to know...

In a clinical evaluation study

96%

of patients considered **ADAPTIC® DIGIT** easy or very easy to remove¹

ADAPTIC® DIGIT

Size	Quantity per Carton	Product code
Small 2cm Ø	10	MAD003
Medium 2.4cm Ø	10	MAD013
Large 2.8cm Ø	10	MAD023
Extra large 3cm Ø	10	MAD042

ADAPTIC® DIGIT TOE

Size	Quantity per Carton	Product code
2.8cm Ø	10	MAD062



Capes Medical
www.capesmedical.co.nz
0800 18 19 19



References

1. Whittaker S. A dressing for the occasion? A comparative trial of two dressings for digit injuries. Professional Nurse Magazine, 1994 Aug. 729-732.
2. ADAPTIC® DIGIT case study, 2812 November 2012.

© Systagenix wound management 2014. Brands marked with ® or ™ are trademarks of Systagenix. All other products referenced herein are acknowledged to be trademarks of their respective owners.