Biodegradable Premier detergent & disinfectant wipes

A single step solution to cleaning and disinfecting surfaces and equipment, proven to kill at least 99.999% of pathogens. With a superior cloth and unsurpassed equipment compatibility, germs don't stand a chance.

Specifications

- Effective for use against COVID and SARS
- · Unparalleled equipment compatibility
- Superior cloth: up to 47% larger than competitors

Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom

Step 1: Peel back the sticky pack cover and take hold of a wipe and pull it out of the pack



Step 2: Open wipe to desired size and re-seal cover to retain moisture.



Step 3: Wipe surface in an 'S' shaped motion from top to bottom.



Step 4: Dispose of wipe. Do not flush or put into a macerator.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.



SKU	WIPES PER PACK	PACKS PER CARTON
RHS210	50	12

Reynard is reassurance.



Australia

44 Hampden Road Artarmon, NSW, 2064 enquiries@reynardhealth.com.au 1300 667 199

New Zealand

PO Box 8470 Havelock North Sales@reynardhealth.com 0508 494 737

Uk

13 White Horse Business Park, Stanford in the vale SN7 8NY uk@reynardhealth.com +44 (0)845 263 7335

Internationa

PO Box 8470 Havelock North Sales@reynardhealth.com +64 (0)6 650 0709