



sutherlandmedical

Introducing **Tubular - Form™**:
The ideal tubular compression
bandage.



COMPRESSION BANDAGES



sutherlandmedical

Tubular - Form™

Tubular Compression Bandage

Tubular - Form™ is ideally designed for the treatment of orthopaedic and vascular conditions.

Tubular - Form™ is used for:

- Treatment of Venous insufficiency requiring low – medium compression.
- Control of Oedema / Swelling of the limbs.
- Soft tissue, joint injuries or rib injuries.
- Application following the removal of casts.

Features:

- 100% Australian Made
- Clinically shown to heal Venous Leg Ulcers with 3 Layer compression therapy (references available).
- Product treated with Unique Low Fray Formula.
- Easy to use Measuring Guides available.
- Easy to use Applicators available.

Measuring Guide for Application

Code	Size	Flat Measure	Application	Roll	Limb Circ**
SM099	AA	3cm	Infant Limbs	10m	< 6cm*
SM100/ SM200^	A	4cm	Infant Limbs	10m	7 – 11cm*
SM101/SM201^	B	6.5cm	Small Limbs	10m	12 – 14cm*
SM102/SM202^	C	7cm	Adult Limbs	10m	15 – 25cm*
SM103/SM203^	D	7.5cm	Large Limbs	10m	26 – 35cm*
SM104/SM204^	E	8.5cm	Adult Legs	10m	36 – 44cm*
SM105/SM205^	F	10cm	Large Legs	10m	45 – 50cm*
SM106/SM206^	G	12.5cm	Large Thighs	10m	51 – 60cm*
SM107	H	14cm	Extra Large Thighs	10m	61 – 70cm*
SM108	J	17.5cm	Small Trunk	10m	71 – 75cm*
SM109	K	21.5cm	Medium Trunk	10m	76 – 80cm*
SM112	K2	27cm	Medium Large Trunk	10m	81 – 84cm*
SM110	L	32.5cm	Large Trunk	10m	85 – 90cm*
SM111	M	37cm	Extra Large Trunk	10m	> 90cm*



*When applying to limbs, measure to fit narrowest part. When applying to other areas of the body measure the affected area

^ 100 series indicates Natural colour 200 series indicates Flesh colour

**Guide is for Medium Compression, for low compression select next size bandage up

Fibre content: Cotton Elastodiene, Polyurethane and Polyester

Sutherland Medical Pty Limited

Unit 2, 195 Chesterville Road Moorabbin Vic 3189

Ph: 1300 664 027

Email: orders@sutherlandmedical.com.au

Web Site: www.sutherlandmedical.com.au

ARTG 132077

Version 4 / Created 18/5/2020